
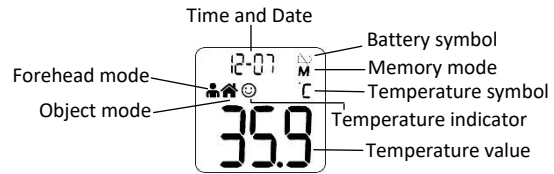


# NON-CONTACT INFRARED THERMOMETER QUICK REFERENCE GUIDE

Device is for Point of Entry Screening Only-NOT for Patient/Resident Use

## Model: DET-306

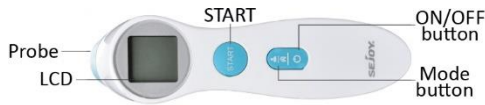

## General:

- Unit requires two AAA batteries – remove battery cover to replace batteries if needed.
- Always store in clean dry place where thermometer will not become excessively cold (-25°C) or hot (35°C)
- Fluctuations in ambient temperature of the environment may reduce accuracy of measurement leading to a false high or low reading (e.g., if placed on window ledge exposed to hot sun, in direct line of air conditioner or fan)
- Place IR thermometer in measuring environment for 30 minutes prior to use for most accurate results.

## Start-up:

- The IR thermometer should be in the same temperature environment it will be used in for approximately 30 minutes prior to taking temperature.
  - Check battery – if battery is low when starting up, replace. The thermometer will turn on in the setup mode, (the date and time does not need to be setup just wait 1 minute for the thermometer to turn off or press and release the on/off button to go through all the steps until the thermometer turns off).
- Setting the temperature to Celsius, With the thermometer off press and hold the start button for 3 seconds and the F symbol will show on the screen, press and release the start button to change to C, press and release the on/off button to exit and save the setting.

## Use:

- Ensure forehead clean, and hair is out of the way for most accurate reading.
  - Note: Cosmetics or sweat on forehead will affect the measurement accuracy.
- Press the on/off button to turn on the thermometer, it will do a self-check and beep, it is now ready to use.
- Aim the thermometer in the centre of the forehead (2-3 cm away from forehead) press and release the start button and you will get the temperature reading.

## Cleaning Instructions:


In the absence of manufacturer's instructions for cleaning, the following IP&C recommendations are to be followed for use and cleaning of Infrared Thermometers:

- Perform hand hygiene prior to use and when stopping use
- Thermometer can be used for multiple people without cleaning/disinfecting in between as long as the thermometer:
  - Does not touch the individual being screened
  - Is being handled by one screener only
  - Is placed on a clean surface (i.e., paper towel, blue pad) between use by a single screener
- Thermometer must be cleaned and disinfected:
  - If the thermometer touches the person being screened
  - If the thermometer becomes contaminated
  - In between use by multiple screeners
  - At end of screening period
- Cleaning and disinfection procedure
  - Obtain an Accel INTERvention disinfectant wipe or OxivirTb wipe – express extra liquid from the wipe before cleaning and disinfecting the device
  - Wipe thermometer with damp wipe, ensuring 1 minute of wet contact time

# NON-CONTACT INFRARED THERMOMETER QUICK REFERENCE GUIDE

**Device is for Point of Entry Screening Only-NOT for Patient/Resident Use**

**Model: IR8805**

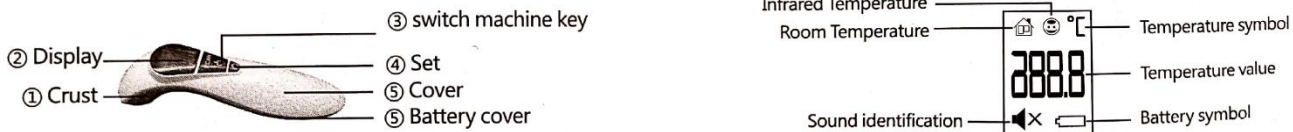

**General:**

- Unit requires two AAA batteries – remove battery cover to replace batteries if needed.
- Always store in clean dry place where thermometer will not become excessively cold (-25°C) or hot (35°C)
- Fluctuations in ambient temperature of the environment may reduce accuracy of measurement leading to a false high or low reading (i.e., if placed on window ledge exposed to hot sun, in direct line of air conditioner or fan)
- Place IR thermometer in measuring environment for 30 minutes prior to use for most accurate results.

**Start-up:**

- The IR thermometer should be in the same temperature environment it will be used in for approximately 30 minutes prior to taking temperature.
  - Check battery – if battery is low when starting up, replace
  - Press “switch key”, unit will conduct a self-check – if there is a problem with the unit an “Err” message will display on the LCD screen
  - After initial start up, unit will enter a “stand-by” state and be ready for use

**Use:**

- Ensure forehead clean, and hair is out of the way for most accurate reading.
  - Note: Cosmetics or sweat on forehead will affect the measurement accuracy.
- Short press the “Setting” for mode conversion – press until unit is in the internal temperature (forehead temperature) measurement mode – this character should appear: “ 😊 ”
- Hold unit 1-5 cm away from forehead and press and hold “switch key” for measurement. Release button to complete measurement.
  - If the measured temperature is higher than 42.1°C then HI will be displayed on the screen.
  - If the measured temperature is lower than 34°C then LO will be displayed.

**Cleaning Instructions:**

In the absence of manufacturer’s instructions for cleaning, the following IP&C recommendations are to be followed for use and cleaning of Infrared Thermometers:

- Perform hand hygiene prior to use and when stopping use
- Thermometer can be used for multiple people without cleaning/disinfecting in between as long as the thermometer:
  - Does not touch the individual being screened
  - Is being handled by one screener only
  - Is placed on a clean surface (i.e., paper towel, blue pad) between use by a single screener
- Thermometer must be cleaned and disinfected:
  - If the thermometer touches the person being screened
  - If the thermometer becomes contaminated
  - In between use by multiple screeners
  - At end of screening period
- Cleaning and disinfection procedure
  - Obtain an Accel INTERvention disinfectant wipe or OxivirTb wipe – express extra liquid from the wipe before cleaning and disinfecting the device
  - Wipe thermometer with damp wipe, ensuring 1 minute of wet contact time